

# LabTEST<sub>BT</sub> PROCEDURE

## STEP 1



TAKE 1 MEMBRANE OUT OF THE WHITE TUBE OR MORE IF YOU NEED TO BLOOD TYPE SEVERAL DOGS.

## STEP 2



ADD 3 DROPS OF BUFFER INTO THE CLEAN TEST TUBE.

## STEP 3



COLLECT 10 $\mu$ L OF WHOLE BLOOD WITH A PIPETTE.

## STEP 4




ADD THE 10 $\mu$ L OF BLOOD INTO THE TUBE FILLED OF BUFFER AND MIX GENTLY.

## STEP 5




INSERT THE MEMBRANE INTO THE WELL TO ALLOW THE MIGRATION.

## STEP 6

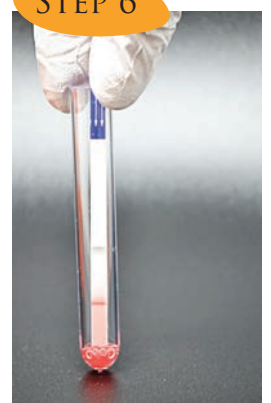

WAIT FOR THE COMPLETE MIGRATION UNTIL THE CONTROL LINE BECOMES VISIBLE.

## STEP 7

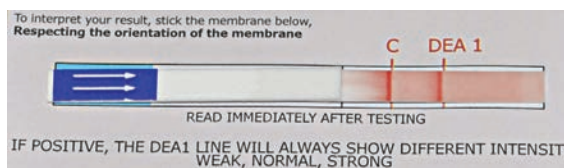

STICK THE MEMBRANE ON THE RESULT FORM TO INTERPRET THE RESULT.

EXAMPLE: IF YOU OBTAIN A LINE IN FRONT OF THE DEA 1, THE DOG IS DEA 1 POSITIVE.

RECORD THIS FORM.