

QuickTEST_{BT} PROCEDURE

1 Alvedia
Blood Typing
Quick Test



1 drop
of blood



2 minutes



STEP 1



WRITE DOWN THE NAME OF THE PATIENT AND THE DATE.

STEP 2



ADD 3 DROPS OF BUFFER INTO THE WELL.

STEP 3



DIP THE BLOOD COLLECTOR STRIP INTO THE BLOOD TUBE OR RUB IT DIRECTLY ON THE DROP OF BLOOD. THE EXTREMITY MUST BE FULLY RED.

STEP 4



DIP THE EXTREMITY OF THE BLOOD COLLECTOR STRIP INTO THE WELL AND MIX GENTLY.

STEP 5



SEPARATE THE PLASTIC DEVICE IN TWO PARTS, BY PULLING OUT THE MEMBRANE HOLDER.

STEP 6



INSERT THE LEGS OF THE MEMBRANE HOLDER INTO THE HOLES BESIDE THE WELL TO ALLOW THE MIGRATION.

STEP 7



ALLOW TO STAND UNTIL THE BLOOD SUSPENSION IS SOAKED UP BY THE MEMBRANE AND THE CONTROL LINE BECOMES CLEARLY VISIBLE.

STEP 8



FIX BACK THE PLASTIC PART IN ORDER TO READ THE RESULT.

SAME PROCEDURE FOR FELINE & CANINE BLOOD TYPING TEST