



# QuickTest BT A+B

Feline result form for blood typing test



Movie procedure

# QuickTest BT A+B



Feline procedure for blood typing test

FIRST-TIME DONOR: Following human good practice, we recommend to perform blood typing twice using 2 kits / 2 blood samples / 2 operators.

Patient name : .....

Identification code : .....

Owner : .....

Blood type :  A  B  AB


Blood typing performed by : .....

Veterinarian

Date : .....

QT lot number : .....

## BLOOD TYPE COMPATIBILITY

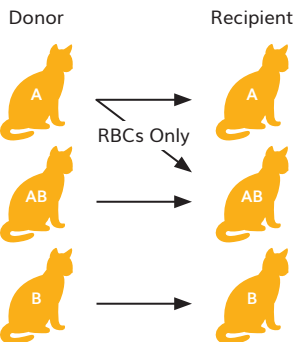

Compatible transfusions are indicated by arrows.

### Before any transfusion

- Always verify AB blood compatibility between donor and recipient.
- For all other blood types : perform a crossmatch test (XM) between donor and recipient.

I agree to be a blood donor

- 
- 
- 
- 
- 
- 
- 
- 

**READ IMMEDIATELY AFTER TESTING**

C = CONTROL LINE

<b>A</b>			<b>WHITE LINE = NEGATIVE RESULT</b>			<b>Please contact us for the «Troubleshooting procedure»</b>
<b>B</b>			<b>WHITE LINE = NEGATIVE RESULT</b>			
<b>AB</b>			<b>WEAK LINE = POSITIVE RESULT</b>			

Alvedia  
Alice Veterinary Diagnostic

www.alvedia.com  
contact@alvedia.com